

**AVAILABLE
EXCLUSIVELY AT
MEDISCA**

**PROVIDE ACCURATE METERED DOSING
DISPENSE WITH ACUCAP CLOSURES**

 **MEDISCA®**

The AcuCap Closure is designed to provide accurate metered dose dispensing of pastes, ointments, creams and gels. When used with compounded medications for topical or transdermal delivery, the disposable AcuCap Closure simply takes the place of an ordinary dispensing container cap.

Features

- Screws onto conventional 16 mm opening dispensing tubes
- Dispenses 0.5 mL doses with each turn of a rotating chamber

Benefits

- Patients and caregivers do not need to measure
- Eliminates the need to repeatedly remove a cap
- Easy for young and elderly patients to use

Specifications

- AcuCap Closures are made of Polypropylene (PP) and High Density Polyethylene (HDPE)
- Manufactured in an ISO 9000 compliant environment

How to use the AcuCap Closure

1. Align the revolving chamber with the tube outlet and squeeze the tube
2. Rotate the chamber 180 degrees
3. Squeeze the tube again



 **1.800.932.1039** www.medisca.com  **1.800.665.6334**

 **MEDISCA®**